

## Instructions for Vapor-Clean Filters

Known or suspected Malignant  
Hyperthermia patients prior to anaesthetic

1. Remove circuit, filters, limb, reservoir bag and sample line and dispose
2. Remove Vaporisers (optional)
3. Flush anaesthetic machine with >10 L/min oxygen for 90 seconds
4. Attach Vapor-Clean filters as labelled one to expired and one to inspired
5. Connect new circuit, limb, reservoir bag and sample line
- 6. MACHINE IS NOW MH SAFE, DO NOT REPLACE VAPORISERS**
- 7. TIVA ANAESTHETIC**
8. Anaesthetise patient, maintaining fresh gas flow at 3 L/min

**NB: Vapor-Clean filters are single use.**

**Using this method, Vapor-Clean filters will last 12 hours**

